


# A Step-by-Step Guide to Burn a CD in eFilm™


Burning a CD enables the user to save selected studies or images to a CD.

**Note:** The study must be on the local hard drive to burn a CD. Select the local exams tab when burning a CD.

## Step 1: Copy an entire study

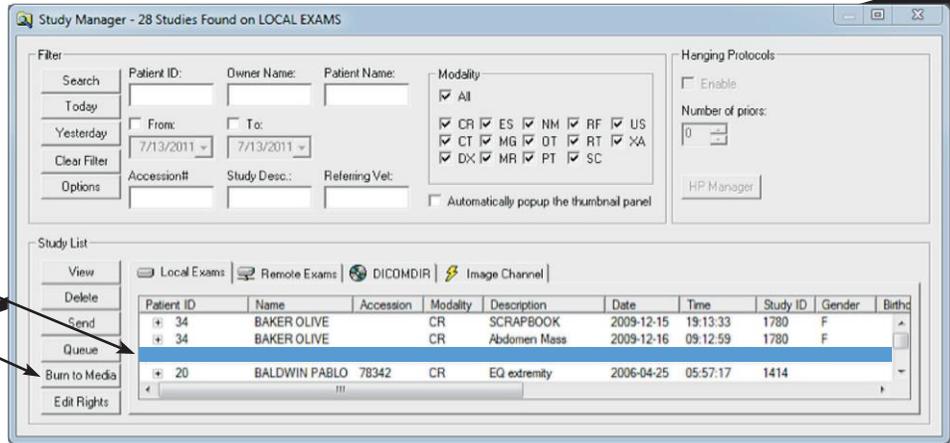


1. Select a Study from the Study Manager List
2. Select Burn CD

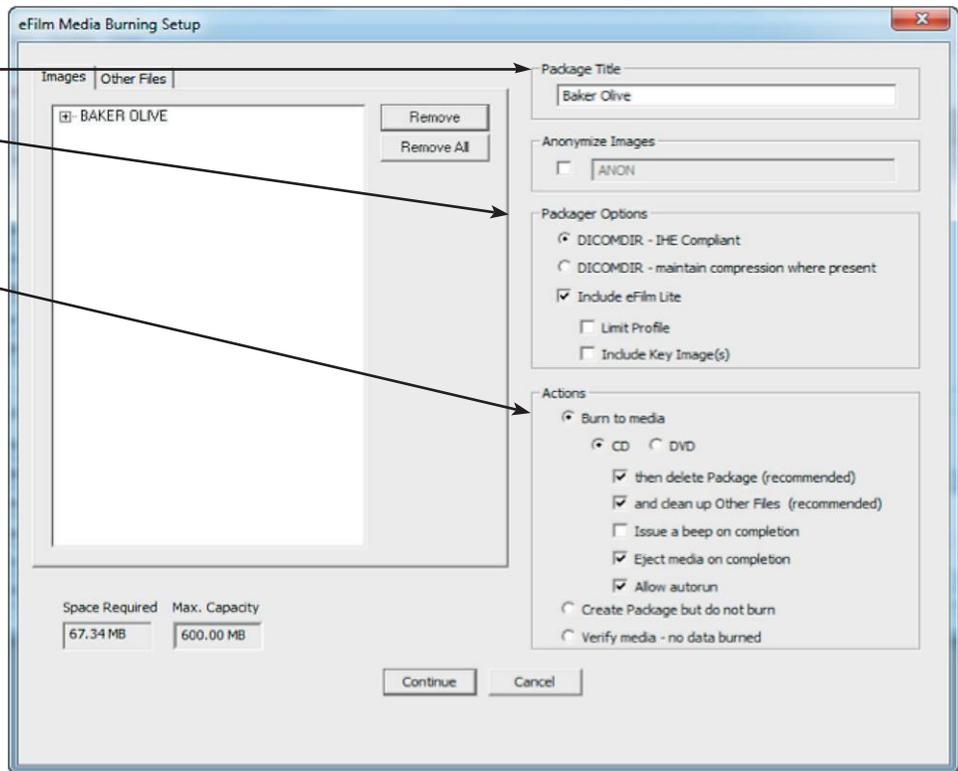


3. Enter CD Title (i.e. owner or patient name)
4. Select CD Package Options:
  - DICOMDIR-IHE Compliant
  - Include eFilm Lite
5. Select CD Burner Options
  - Burn to media CD
  - then, delete Package (recommended)
  - Clean up Other Files (recommended)
  - Eject Media on Completion
  - Allow autorun
6. Insert CD and select Continue

## Copy multiple studies in sequence

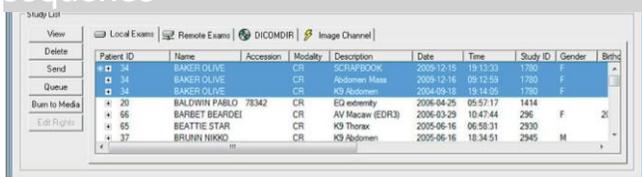

## Copy multiple studies not in sequence

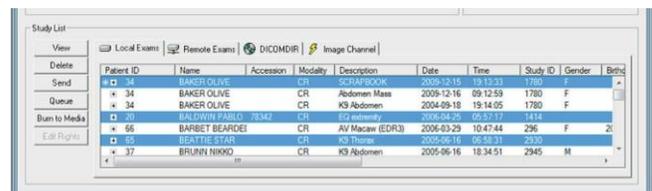

# A Step-by-Step Guide to Burn a CD in eFilm™

-5 above

# A Step-by-Step Guide to Burn a CD in eFilm™

**Note:** Other applications are not available during the burn process.

## Copy selected images

1. Search Study Manager List for desired study.
2. Open study
3. Select images to copy
4. Select Utility
5. Scroll down to create CD
6. Repeat 'Step 1' sequence



Technical Support: 800.268.5354, option 3  
For after hours emergencies use the same number, option 3, and select option 1

YOUR IMAGE IS OUR BUSINESS.



a VCA ANTECH company

800.268.5354 [soundeklin.com](http://soundeklin.com)