

TruDR
AIS Sending Document

Sending consultation:

- 1) In TruDR, highlight the study you want to send and look in the menu bar to find the AIS button , click on it to bring up the consultation screen (see example of screen below)

New Consultation

Consultation Information

* - Required

Selected Practitioner * : VetFirstName VetLastName of DarkHorse Demonstration Center

Patient Name *	Owner Last Name *	Owner First Name *
<input type="text" value="AIS Test 2"/>	<input type="text" value="Jackson"/>	<input type="text" value="Action"/>

Species *	Breed	Other Breed (if not in breed list)
<input type="text" value="Canine"/>	<input type="text" value="Alaskan Malamute"/>	<input type="text"/>

Date of Birth * 1999 September 19	Color <input type="text"/>	Sex * Male/Neutered
Chip No. <input type="text"/>	Case No. AIST2	Owner Phone <input type="text"/>

Priority & Response Time hrs <-- Select 2 for STAT case

Select Specialty *

Select specialties by moving a highlighted specialty from left to right. Order the specialties by using the up-down controls below the right select field.

Available Specialties		Current Specialties
<ul style="list-style-type: none">CardiologyCardiology with VerbalClinical PathologyComputed TomographyDentistryDermatologyECG - Event MonitorECG - Holter MonitorECG - PreopECG - RoutineECG - StatMagnetic Resonance ImagingMedicineMedicine with VerbalOncology	<input type="button" value="←"/> <input type="button" value="→"/> <input type="button" value="<<"/> <input type="button" value=">>"/>	<input type="text"/> <input type="button" value="v"/> <input type="button" value="^"/>


Wt: <input type="text" value="31"/> lbs	Temperature: <input type="text"/> C	Pulse: <input type="text"/> bpm	Respiration: <input type="text"/> per minute
---	-------------------------------------	---------------------------------	--

History * (Not required for PenHIP)

History is very important and is a required field

AIS Sending Document

- 2) Now fill in any information required for the consult (Practitioner, Specialty History, Descriptions of the case, Practitioner Comments, etc...) and click on the "Submit" button at the bottom of the page.

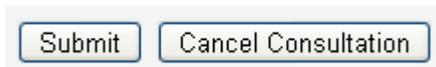

- 3) Once the study has been submitted you should see a "Consultation Successfully Saved" message, as seen in the example below.



- 4) Close the AIS page to finish.

Note: You may log onto AIS to verify that the exam sent. Reports and images will remain on AIS for easy access from any Internet connection.

Your report will be faxed and an email will be sent. Turnaround times

5am – 5pm 4hrs

5pm – 5am by 9am

You can always request a STAT case.